
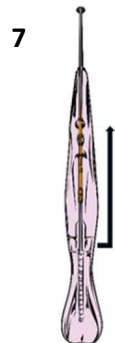
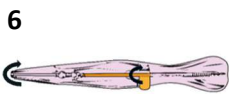
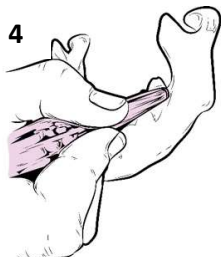
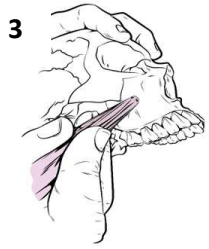
BONEGAINER

Instruction for Use (IFU)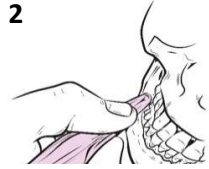

Field of application

The products may only be used by trained and qualified personnel. The surgeon must determine the patient's suitability for bone transplantation. The surgeon must take all necessary precautions to prevent the risks of infection and complications associated with bone transplantation procedures.

Intended purpose

The intended purpose of the BoneGainer is the minimally invasive harvesting of autologous (the patient's own) cortical bone material for dental augmentation and the regeneration of bone defects.

Indication

Intraoral harvesting of autologous cortical bone chips for bone augmentation, particularly in dentistry.

Contraindications

Acute infections, uncontrolled systemic diseases such as diabetes or cardiovascular disorders, severe allergic reactions, certain immunosuppressive therapies, bisphosphonate therapies, severe osteoporosis, poor oral hygiene, smoking, as well as severe psychological disorders that would complicate follow-up care.

Fig. 1

Scraping position with the index finger in the upper grip recess

Fig. 2

Index and middle finger in the lateral grip recess

Fig. 3

Scraping position with the index finger in the upper grip recess and the index and middle finger in the lateral grip recess

Fig. 4

Scraping position with the thumb in the upper grip recess and the index and middle finger in the lateral grip recess

Fig. 5

Scraping position with the thumb on top

Fig. 6

Rotating the lever to the other locking position changes the blade side at the working tip

Fig. 7

Pushing the slider forward causes the blade shaft to slide out

Reportable incidents

Please report all serious incidents or near-incidents that have occurred in connection with the product to PalDent Tech GmbH using the contact details provided in these instructions for use, as well as to the competent authority in your country.

- Preparation:** After opening the package, place the BoneGainer on a sterile tray and maintain sterile working conditions throughout the entire procedure.
- Ergonomic handling:** Use the grip recesses for ergonomic finger placement (Figures 1 to 5).
- Bone processing:** Place the working tip with the blade on the bone to be scraped and perform scraping movements (Figures 1 to 5). The collected bone chips pass through the gap between the blade and the handle into the chamber.
- Removing bone chips:** The collected bone chips are visible in the chamber. Release the horizontal lever on the underside of the BoneGainer from its locked position and push the slider toward the instrument tip (Figures 6 and 7). This extends the long blade shaft fully, allowing the bone chips to be removed (Figure 7).
- Reset:** Before disposal, slide the slider back to its original position and lock the lever laterally into place. The entire blade shaft will then be back in its original position inside the chamber.

Important information	
	Please read these instructions for use carefully before application and keep them easily accessible for the user/operator.
	Please carefully read the warnings marked with this symbol. Improper use of the products may lead to serious injury to the patient, the user, or third parties.
	The products are intended for single use only and are not designed for reuse.
STERILE R	The product is supplied STERILE. It has been sterilized using a radiation sterilization process.
	The products are not intended for re-sterilization.
	Before using the products, ensure that the expiration date has not been exceeded.
	Products with damaged packaging must not be used.
Safety instructions	
	The products are intended for single use only and must not be reused.
Storage	
	The products must be protected from direct sunlight during storage.
	The products must be stored in a dry place.
Disposal	
After use, dispose of the product in the designated special container for medical waste in accordance with applicable regulations.	

Explanation of symbols	
MD	Medical device
REF	Article number
LOT	Batch code
QTY	Quantity
UDI	Unique Device Identifier (UDI)
	Consult instructions for use
	Do not reuse
	Do not re-sterilize
	Do not use if packaging is damaged

Explanation of symbols	
	Manufacturer
	Use-by date
	Single sterile barrier system
	Simple sterile barrier system with outer protective packaging
	Keep dry
	Protect from direct sunlight
STERILE R	Sterilized using irradiation
R_xonly	According to U.S. federal law, this product may only be sold by or on the order of a physician or dentist.

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